
*Glucoplus*TM
FastTouch[®]
LANCING DEVICE

Instructions for Use

The GlucoPlusTM FastTouch[®] Adjustable Lancing Device provides an easy, safe way to obtain capillary blood samples needed for blood glucose monitoring. The adjustable tip on the Lancing Device allows you to choose the best depth of skin penetration. To obtain a drop of blood, first wash your hands with warm, soapy water. Dry your hands thoroughly before using the Lancing Device.


1. Remove the depth adjustment cap. Insert a lancet into the lancet holder and push down until it is fully seated.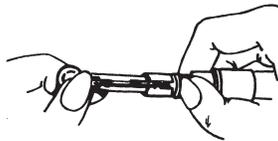

2. Twist off the lancet's protective disk. Make two full turns to ensure that the lancet separates from the protective disk.


3. Replace the Lancing Device cap and set the puncture depth to the desired number. There are 5 levels of skin penetration offered. To select a desired depth, move the number to match the arrow.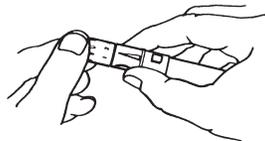


To select the best depth:

- 1-2 for soft or thin skin
- 3 for average skin
- 4-5 for thick or calloused skin


4. Pull back the cocking mechanism until it makes a click, and then release. If it does not click, the device may have been cocked when the lancet was inserted.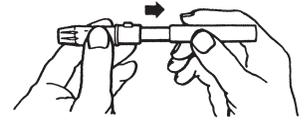

5. Release the sleeve. It will automatically move back to its original position near the trigger and lift up the device.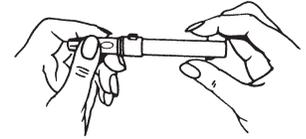

6. Position the end of the Lancing Device against the tip of the finger. Press the trigger button, and lift up the Lancing Device away from the finger after the puncture is complete.

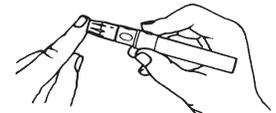7. Set aside the Lancing Device and wait a few seconds for a blood drop to form. The flow of blood will be helped by keeping the hand warm, by lowering the hand to waist level, and by gentle massage of the finger.

8. For safety reasons, and to prevent cross-contamination, discard the used lancet into an appropriate sharps or biohazard container. Never allow another person to use a lancet that has already been used. If the Lancing Device is to be used by another person, the unit must be provided with a new tip and sterile lancet.

9. Lancet removal: Unscrew the Lancing Device tip. Carefully stick the protective disk cover back on the used lancet. Grip the holder firmly and pull the lancet safely out of the holder. Screw the tip back in the Lancing Device.

Taking care of your Lancing Device

Use alcohol or soap and water to wipe the outside of the Lancing Device. For more complete cleaning, remove the tip, wash it in warm water and rinse well. It can be disinfected with 70% rubbing alcohol. (Hospitals should consult their own infection control protocol.)

Limited Warranty

Glucoplus warrants that the Lancing Device will function effectively for a period of five (5) years from date of purchase. A malfunctioning device should be returned to:

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Glucoplus Inc.
GlucoPlus Inc.